



The Dental Pro's Perfect Fit™ Guarantee

Thank you for choosing Dental Prosthetics, Inc. as your precision Authorized Lava Milling Center. Your Lava copings will be designed and milled to very exact specifications ensuring a *Dental Pro's Perfect Fit™*. Please take a moment to review these guidelines to validate the guarantee and to ensure optimal results and the quickest turn-around time.

How to Ensure the Most Precise Fit Possible and Fastest Turn-around Time

1. Ensure that chamfer prep matches 3M ESPE Prep Guide specifications.
2. Double check that each bridge has a proper path of insertion with all margins visible simultaneously.
3. Use neutral, buff or white colored die stone.
4. Please do not mark red line on margin.
5. Please do not use die paint, sealer, superglue, etc. (to ensure better fit).
6. All undercuts must be blocked out.

Did you send the following items with your case?

- Dental Pro's Lab Slip with shade and any special instructions.
- Single Unit?** Only include the die.
- Bridge?**
 - Removable, pinned dies
 - Removable adjacent teeth
 - Removable pontic
 - Based model should be removed from metal-based articulator. Plastic articulators are acceptable. This will ensure models will fit into scanner.
 - Putty matrix bite that stays on the master model and shows the opposing teeth in the matrix.
- A bite registration, if available from Doctor.
- If using an Adjustable Articulator, please only send the models on the mounting brackets (no articulator included).

NOTE – Your case will be returned using DHL next-day shipping at no additional charge!